N° 7308

PX Gold LF Star+

PX DENTAL SA

|                |                             |
|----------------|-----------------------------|
| JAUNE CLAIR    | ALLIAGE DENTAIRE UNIVERSEL  |
| HELL GELB      | DENTAL UNIVERSALLEGIERUNGEN |
| GIALLO CHIARO  | LEGA DENTALE UNIVERSAL      |
| LIGHT YELLOW   | DENTAL UNIVERSAL ALLOY      |
| AMARILLO CLARO | ALEACIÓN DENTAL UNIVERSAL   |

% = Au: 38.0, Pd: 17.0, Ag: 36.0, In: 8.9, Ir: < 1%

12.4 g/cm<sup>3</sup>

1020 - 1075°C

16.9/17.4 µm/m° C

ISO 9693 /22674

Fabriqué par / Hergestellt von / Fabricato da /  
Made by / Fabricado por

PX DENTAL SA

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### 1. Waxing up, spruing, investing

- Follow the general prescriptions for bonding alloys.
- The ultimate thickness of the cast coping should not be less than 0.3 mm.
- Use only a high-heat phosphate-bonded investment material. For the alloy PX GOLD LF STAR + we recommend to use the PX Extravest investment.

### 2. Preheating

- Observe manufacturer's recommendations with regard to setting times, temperature levels, etc.
- Keep the cylinder at 750° C for a soaking period of 30-45 min.

### 3. Crucibles and casting buttons

- Use a ceramic crucible.
- Add at least 1/3 of new alloy. When remelted casting buttons should be perfectly clean.

### 4. Melting

- For PX GOLD LF STAR + casting temperature is 1225° C with a ceramic crucible.
- The use of crystallized boric acid is recommended.
- Bench cool the casting ring.

### 5. Treatment of the casting

- Devesting: by blasting (grain size of 50-125 µm).
- Following casting or soldering, the frame may be pickled.
- Preparing the framework and cleaning: trim the framework preferably using carbide burs or fine carborundum stones at low speed another blasting (grain size of 50-125 µm) and clean the frameworks in an ultrasonic unit or with a steam cleaner.

### 6. Oxidation

- Due to composition, this alloy not needs oxidation. Directly, blasting (grain size of 90-125 µm). Do not pickled, just clean the framework in an ultrasonic unit or with a steam cleaner

### 7. Porcelain application

- Porcelain applications are subject to the manufacturer's instructions. For a better aesthetic use opaque pastes
- PX GOLD LF STAR+ is compatible with low fusing ceramics (like Duceragold). Please observe the required coefficient of expansion.

### 8. Soldering

- Pre-bond: use PX SOLD LF 950 solder at 950° C with a flux.
- Post-bond: use PX SOLD 700 solder at 750° C with a flux

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