N° 7030

PX PREMIUM S

CE 1250

PXDENTAL SA

JAUNE CLAIR	ALLIAGE DENTAIRE CÉRAMO MÉTALLIQUE
HELLGELB	DENTAL AUFBRENNLEGIERUNGEN
GIALLO CHIARO	LEGA DENTALE PER CERAMICA
LIGHT YELLOW	DENTAL BONDING ALLOY
AMARILLO CLARO	ALECIÓN DENTAL CERAMO-METALLICA

% = Au: 84.4; Pt: 7.9; Pd: 4.6, In: 2.6, Ir, Cu: < 1%

18.1 g/cm³
1130 - 1190°C
14.2 /14.4 µm/m° C

ISO 9693

Fabriqué par / Hergestellt von / Fabricato da /
Made by / Fabricado por

PX DENTAL SA

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1. Waxing up, spruing, investing

- Follow the general prescriptions for bonding alloys.
- The ultimate thickness of the cast coping should not be less than 0.3 mm.
- Use only a high-heat phosphate-bonded investment material. For the alloy PX PREMIUM S we recommend to use the PX Extravest investment.

2. Preheating

- Observe manufacturer's recommendations with regard to setting times, temperature levels, etc.
- Keep the cylinder at 850° C for a soaking period of 30-45 min.

3. Crucibles and casting buttons

- The use of a ceramic crucible is recommended.
- Add at least 1/3 of new alloy. When remelted casting buttons should be perfectly clean.

4. Melting

- For PX PREMIUM S casting temperature is 1340° C with a graphite crucible, 1390° C with a ceramic crucible
- The use of crystallized boric acid is recommended.
- Bench cool the casting ring.

5. Treatment of the casting

- Devesting: by blasting (grain size of 50-125 µm).
- Following casting or soldering, the frame may be pickled.
- Preparing the framework and cleaning: trim the framework preferably using carbide burs or fine carborundum stones at low speed another blasting (grain size of 50-125 µm) and preceding oxidation clean the frameworks in an ultrasonic unit or with a steam cleaner.

6. Oxidation

- 10 min under vacuum at 950° C.

7. Porcelain application

- Porcelain applications are subject to the manufacturers instructions.
- PX PREMIUM S is compatible with most known porcelain brands (high fusing).

8. Soldering

- Pre-bond: use PX PRESOLD 1100 solder at 1100° C with a flux.
- Post-bond: use PX POSTSOLD 800 solder at 850° C with a flux.

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