# PXDENTAL SA

JAUNE CLAIR	ALLIAGE DENTAIRE CÉRAMO MÉTALLIQUE
HELLGELB	DENTAL AUFBRENNLEGIERUNGEN
GIALLO CHIARO	LEGA DENTALE PER CERAMICA
LIGHT YELLOW	DENTAL BONDING ALLOY
AMARILLO CLARO	ALECION DENTAL CERAMO-METALLICA

% = Au: 78.8, Pt: 9.8, Pd: 7.8, In: 3.5, Ir: < 1%

17.7 g/cm<sup>3</sup>

1120-1240° C

14.0/14.2 µm/m° C

ISO 9693

Fabriqué par / Hergestellt von / Fabricato da /  
Made by / Fabricado por

**PX DENTAL SA**

Champs-Montants 16 a, 2074 Marin

Switzerland

T. +41 (0)32 924 21 20 F +41 (0)32 924 21 29

www.pxdental.com

**PXDENTAL SA**  
CH - 2074 MARIN

## Mode d'emploi

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**1. Waxing up, spruing, investing**

- Follow the general prescriptions for bonding alloys.
- The ultimate thickness of the cast coping should not be less than 0.3 mm.
- Use only a high-heat phosphate-bonded investment material. For the alloy PX PREMIUM IDEAL we recommend to use the PX Extravest investment.

**2. Preheating**

- Observe manufacturer's recommendations with regard to setting times, temperature levels, etc.
- Keep the cylinder at 850° C for a soaking period of 30-45 min.

**3. Crucibles and casting buttons**

- The use of a ceramic crucible is recommended.
- Add at least 1/3 of new alloy. When remelted casting buttons should be perfectly clean.

**4. Melting**

- For PX PREMIUM IDEAL casting temperature is 1390° C with a graphite crucible, 1440° C with a ceramic crucible
- The use of crystallized boric acid is recommended.
- Bench cool the casting ring.

**5. Treatment of the casting**

- Devesting: by blasting (grain size of 50-125 µm).
- Following casting or soldering, the frame may be pickled.
- Preparing the framework and cleaning: trim the framework preferably using carbide burs or fine carborundum stones at low speed another blasting (grain size of 50-125 µm) and preceding oxidation clean the frameworks in an ultrasonic unit or with a steam cleaner.

**6. Oxidation**

- 10 min under vacuum at 950° C.

**7. Porcelain application**

- Porcelain applications are subject to the manufacturers instructions.
- PX PREMIUM IDEAL is compatible with most known porcelain brands (high fusing).

**8.Soldering**

- Pre-bond: use PX PRESOLD 1100 solder at 1100° C with a flux.
- Post-bond: use PX POSTSOLD 800 solder at 850° C with a flux.

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