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EN Important instructions for use and processing of the Dental Alloys

The selection, processing, temporary or permanent cementation, and repair of dentures must be performed by a professional Dentist or Dental Technician. It can generally be assumed that the treatment is known, since dental alloys have been part of the basic training in dentistry / dental technology for several decades. Alloys are products for professional use only.

Qualification

The knowledge of a professional dentist and dental technician is required. The instructions for use must be available at all times and must be read in full before the first use. The medical device may only be manufactured by qualified technical personnel.

These instructions for use are guidelines and are based on the current state of the art.

- a) The indications/contraindications for use must be observed.
- b) The instructions do not release the user from the obligation to personally test the product for suitability for the intended use, indication and use on the patient. The indication and use of the products must be decided by the treating physician/dentist, e.g. on the basis of an X-ray and a qualified diagnosis.
- c) Modification or mixing of alloys is beyond our control and is the responsibility of the user. Any liability for damages and consequential damages caused in this process is excluded.
- d) In the case of patients with an existing allergy to one or more elements of the alloys, these alloys should not be used in the allergic patient.

Traceability

Traceability of the medical devices must be guaranteed by the customer/dental technician. The dental technician must indicate the corresponding LOT numbers on his documents and/or pass them on to the end customer. In case of mixing of LOTs of the same alloy, all lot numbers must be included in the end customer's traceability document.

Mixing of different alloys or alloys of the same type (e.g. conventional) is not allowed!

Selection / Indications

The indications for the various prosthetic applications are given in the alloy table and the instructions for use.

Intended use

Fixed and removable dental prosthesis

Expected clinical benefit

Restoration of occlusion, mastication and improvement of appearance.

Product description

Precious dental alloys according to ISO 22674 are characterized by their fine-grained cast structure, high corrosion resistance, biocompatibility and easy processing. Due to the high content of precious metals Au, Pt and Pd, the alloys can be soldered without any problems; they are also very suitable for connection casting in case of combined work.

Contraindications

Do not use in case of known hypersensitivity to one or more of the metals contained in the alloy. Avoid occlusal and proximal contacts of different alloys. Unfavorable periodontal environment.

Side effects

For a patient who is allergic to one or more elements of an alloy, that alloy should not be used. For a patient who has a suspicion of being allergic to one or more elements of an alloy, an allergological test will be performed beforehand to clarify his uncertainty and to demonstrate that the alloy can be used if he does not show an immune response.

Target group

The treating dentist should provide guidance on the care and need for dental restoration, based on lifestyle, age or family history.

The entire population can be affected by the need for a dental restoration. Precious alloys are well suited to different clinical situations.

The dentist will decide which type of restoration will be performed and with which alloy.

There is no particular patient profile or contraindication for a given population. However, patients are usually elderly.

After labor, the device is placed only in the oral cavity and is in contact with the mucosa. There is NO subgingival implantation. Tissue contacts are: possibly gingiva, dentin, teeth and saliva.

IMPORTANT IN CASE OF INCIDENT: The user or patient must report any serious incident in connection with the medical device and this should be notified to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Cleaning, disinfection, sterilization

All our alloys are semi-finished products and must be processed / modified by the user. Therefore, they are not supplied sterile, disinfected or specially cleaned.

Once the parts are finished, they must be cleaned before being used. Disinfect the parts only with / in appropriate solutions.

All parts must be disinfected before use with a high-level disinfectant. Follow the manufacturer's instructions for use regarding dosage and exposure time.

The disinfectant must be suitable for cleaning and disinfecting dental prosthetic components, must be compatible with the materials of the products to be cleaned and disinfected and must have proven disinfecting effectiveness.

We recommend the use of orthophthalaldehyde solution, strictly follow the manufacturer's instructions for use.

Expiration date

Lifetime: at least 10 years. In principle, the life span of precious metal prosthesis is not limited, but it must be checked periodically, preferably annually, by the dentist.

Under good health and dental hygiene conditions, this type of prosthesis (precious metal) can last indefinitely. Problems occur when the gums recede and the teeth become loose, or when debris gets under the denture.

Gold never reacts with oxygen (one of the most active elements), which means it does not rust or tarnish. The tarnish of gold is very fine and manifests itself as a darkening of the reflective surfaces. Due to their unique properties, such as physical and chemical resistance, precious alloys do not have an expiration date.

Shelf-life: at least 10 years. Precious metal alloys that have been preserved and stored in their original packaging can in principle be used without restriction. They do not degrade and retain their original physical, mechanical and chemical properties.

Precautions for use

These new instructions for use supersede all previous instructions for use.

The instructions for use must be followed exactly. The manufacturer accepts no liability for damage resulting from failure to follow the following instructions for use

Treatment, maintenance and repair in the mouth must be carried out by a specialist (usually a dentist).

Only original alloys may be used.

During treatment in the laboratory and dental practice, as indicated

- Wear dark glasses and protective gloves during casting.

- For casting add at least 1/3 new alloy. The reused alloy must be completely clean and free of coating and flux residues.

- During pickling, wear protective glasses and gloves against acid splashes and avoid inhaling the fumes by wearing a mask.

- During processing with rotating instruments, protect your eyes with goggles, wear a protective mask and work behind a vacuum.

- Precautionary measures must be taken to prevent the patient from swallowing or inhaling the product during treatment (rubber dam, etc.)

- The products must be cleaned and disinfected before being placed in the mouth.

Clarifications/complaints can only be accepted if batch numbers are indicated.

Visits to the dentist

Periodic check-ups, preferably annual, are necessary.

Restoration care

Alloy dentures should be cleaned at least once a day. After cleaning with detergents, always rinse thoroughly with water.

If the customer cleans the restoration with abrasive brushes and aggressive toothpastes, premature wear of the functional parts may occur.

Residual risks

Please pay particular attention to the above-mentioned points. Incorrect use of the alloys (incorrect indication) or use in patients who are allergic to certain alloy components may lead to problems.

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