GP-Ball

Mode d'emploi

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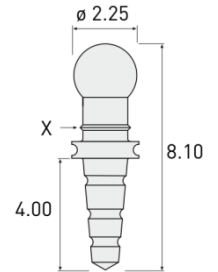
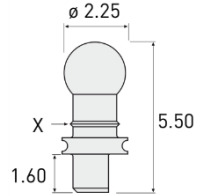
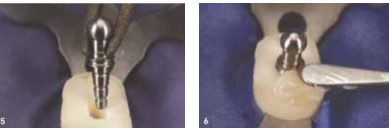
GP-Ball

Instructions for Use Modo de empleo

Produit for Professional use only. Before begin please read "Important Notes on Use and Operation of Dental Attachments"

The long version of the GP-Ball root anchor is used for the direct method, while the short version is used for the indirect method.
X Upper limit for the composite cap.

Indication

Treatment of the roots of anterior teeth, premolars and molars. Anchoring of prostheses also in social medicine and gerontology.

Description

GP-Ball Root Anchors can be used to fabricate a composite root cap with a spherical anchor in an appropriate manner due to their simple handling. GP-Ball Root Anchors without threads are cemented into the tooth root without tension and serve as retention for a prosthesis. The matrices of the following systems are recommended for use: Ecco, Sfera and Tima.

Contraindications

Prostheses with unilateral extension saddle, without cross connection

Indirect method

Fig. 1
Preparation of the root canal inlay

Fig. 2
Indirect composite coping made in the dental laboratory

Picture 3
Cemented indirect composite coping

Fig. 4
In situ prosthesis

Direct method

Photo 5
GP-Ball trial and cementation

Fig. 6
Direct construction of the composite cap

Fig. 7
Final composite coping

Fig. 8
In situ prosthesis

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