N° 7308

PX Gold LF Star+

**PX Dental SA**Bvd Eplatures 42, c/o PX group
CH-2304 La Chaux-de-Fonds**PX DENTAL**

JAUNE CLAIR	ALLIAGE DENTAIRE UNIVERSEL
HELL GELB	DENTAL UNIVERSALLEGIERUNGEN
GIALLO CHIARO	LEGA DENTALE UNIVERSAL
LIGHT YELLOW	DENTAL UNIVERSAL ALLOY
AMARILLO CLARO	ALAECIÓN DENTAL UNIVERSAL

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Instructions for use

Indications

Central or multisurface inlays, ¾ crowns, full crowns, Short and long span bridges, milled telescope crowns, implant suprastructures.

Contraindications

Do not use in known hypersensitivity to one or several metals contained in the alloy, Avoid occlusal and proximal contacts of different alloys, Unfavorable periodontal environment

1. Waxing up, spruing, investing

- Follow the general prescriptions for bonding alloys.
- The ultimate thickness of the cast coping should not be less than 0.3 mm.
- Use only a high-heat phosphate-bonded investment material.

For the alloy PX GOLD LF STAR + we recommend to use the PX Extravest investment.

2. Preheating

- Observe manufacturer's recommendations with regard to setting times, temperature levels, etc.

- Keep the cylinder at 750° C for a soaking period of 30-45 min.

3. Crucibles and casting buttons

- Use a ceramic crucible.
- Add at least 1/3 of new alloy. When remelted casting buttons should be perfectly clean.

4. Melting

- For PX GOLD LF STAR + casting temperature is 1225° C with a ceramic crucible.
- The use of crystallized boric acid is recommended.
- Bench cool the casting ring.

5. Treatment of the casting

- Devesting: by blasting (grain size of 50-125 µm).
- Following casting or soldering, the frame may be pickled.
- Preparing the framework and cleaning: trim the framework preferably using carbide burs or fine carborundum stones at low speed another blasting (grain size of 50-125 µm) and clean the frameworks in an ultrasonic unit or with a steam cleaner.

6. Oxidation

- Due to composition, this alloy not needs oxidation. Directly, blasting (grain size of 90-125 µm). Do not pickled, just clean the framework in an ultrasonic unit or with a steam cleaner

7. Porcelain application

- Porcelain applications are subject to the manufacturer's instructions. For a better aesthetic use opaque pastes
 - PX GOLD LF STAR+ is compatible with low fusing ceramics (like Ducergold).
- Please observe the required coefficient of expansion.

8. Soldering

- Pre-bond: use PX SOLD LF 950 solder at 950° C with a flux.
- Post-bond: use PX SOLD 700 solder at 750° C with a flux

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