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EN Important Notes on Use and Operation of Dental Attachments

Qualification

The selection, processing, activation and deactivation, and repair of attachments should be **performed by a professional Dentist or Dental Technician**. It can generally be assumed that the processing is known, since accessories such as root posts, anchors, attachments and bars, etc. have been part of the basic training in dentistry/dental technology for several decades. The instructions for use must be available at all times and read completely before first use. These operating instructions are a guide and are based on the current state of the art.

- Only original parts and auxiliary tools may be used for treatment. The use of components that are not part of the system may impair the function of the accessories and excludes any warranty or replacement by the manufacturer.
- These treatment instructions are guidelines and are based on the current state of the art. The indications/restrictions for use must be observed.
- The instructions do not release the user from the obligation to personally test the product for its suitability for the intended use, indication and use on the patient. The indication and use of the products must be decided by the treating physician/dentist, e.g. on the basis of an X-ray and a qualified diagnosis.
- The processing of attachments is beyond our control and is the responsibility of the user. Any liability for damages and consequential damages caused in this process is excluded.
- In the case of patients with an existing allergy to one or more elements of the attachment materials, these attachments should not be used.

Traceability

Traceability of the medical devices must be guaranteed by the customer/dental technician. The dental technician must indicate the corresponding LOT numbers on his documents and/or pass them on to the end customer.

Selection / Indications

The indication of the different accessories/systems is shown in the brochure and the operating instructions.

general rule: Always choose the strongest, largest and most abrasion-resistant fastener variant. Use several attachments whenever possible (observe the instructions). Support on several roots, pillars. Rigid attachments must always be protected by circumferential milling. Exceptions are mentioned in the leaflets. Observe the degrees of freedom of the attachments.

Intended use

The components are intended for use in prosthetic restorations on root canopies to support procedures in the dental clinic or in the laboratory.

Expected clinical benefits

Restoration of occlusion, masticatory function and improvement of esthetics.

Description of the product

Anchorage or attachment is the name given to a prosthetic retaining element consisting of a matrix (outer part) and a patrix (inner part). The patrix is soldered onto a custom-made root post coping. The matrix is cemented into the removable denture or polymerized directly. The root post coping itself consists of a cast metal coping with a root post. As a high-quality solution, the prefabricated root post is integrated into the metal coping by means of a sprue. As an economical alternative, a burn-out. Retentive restorations snap audibly into place and give the patient a secure feeling for a correctly seated, removable denture.

Contraindication

The contraindication/restriction of use for each attachment system is indicated in the brochure or instructions for use.

Placement

Observe the insertion direction of the prosthesis. If multiple attachments are used, always ensure parallelism / same direction of insertion. Anchors: If multiple anchors are used, position all dies in the same axis.

IMPORTANT: Also align all dies in the same axis.

Attachments: In the case of 2 movable attachments (with transverse lock and free end saddle), both attachments must be equally aligned in all three dimensions.

Bars: Ovoid bars are primarily shown at 13-23 / 33-43. Rigid bars can have several pillars.

Side effects

In patients with an existing allergy to one or more elements of the product materials, this product must not be used. In patients with a suspected allergy to one or more elements of the product materials, this product may only be used after prior allergological clarification and proof of the non-existence of an allergy. Auxiliary instruments may contain nickel. No side effects are known when used properly.

Target group

Depending on lifestyle, age or family history, the dentist treating the patient should advise on the care and necessity of dental rehabilitation.

The entire population may be affected by the need for tooth restoration. Attachments are well suited for a variety of clinical situations.

The dentist decides what type of restoration to perform and with what anchorage.

There is no specific patient profile or contraindication for a particular population. Nevertheless, the patients are usually elderly people.

After the work is done, the appliance is only placed in the oral cavity and comes into contact with the mucosa. There is NO subgingival implantation.

Tissue contacts are: possibly the gums, dentin, teeth and saliva.

IMPORTANT IN CASE OF INCIDENT: The user or patient must report any serious incident that occurs in relation to the medical device and this should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Cleaning, disinfection, sterilization

Most attachments are semi-finished products and must be processed/modified by the user. Therefore, not all attachments are supplied sterile, disinfected or specially cleaned.

If the attachments are not modified or processed, they must be cleaned before use. Only disinfect the attachments with / in suitable solutions. Never sterilize plastic accessories, do not use acids, do not sandblast, do not heat with steam.

The disinfectant must be suitable for cleaning and disinfecting dental prosthetic components, must be compatible with the materials of the products to be cleaned and disinfected, and must have a proven disinfecting effect.

We recommend the use of an orthophthalaldehyde solution, strictly follow the manufacturer's instructions for use.

Expiration date

The durability of a prosthetic element made of a precious metal alloy should be checked by the dentist at regular intervals, preferably annually.

With good dental health and hygiene, this type of denture (precious metals) can last indefinitely. Problems occur when the gums recede and the teeth become loose, or when debris gets under the denture.

Gold never reacts with oxygen (one of the most active elements), which means it will not rust or tarnish. Tarnishing of gold is very subtle and is indicated by a darkening of the reflective surfaces. Due to their unique properties, such as their physical and chemical resistance, attachments made of precious metals have no expiration date.

Activate, deactivate

Carefully activate/deactivate the fasteners with the original tools.

Replacing parts

Replace replaceable parts only with original parts. In the case of adjustment parts, first check whether they are indicated (check the wear and dimensions of the parts). For threaded parts: when screwing in, first turn to the left until the threads engage, then tighten to the right. When available, always use a torque wrench.

Bars: remove old PEEK tabs, push in new PEEK tabs.

Relining

Make the model: Insert the model pin into the cavity and make the working model.

Wear and tear of the fixation parts

The fixation parts can wear out. The reasons for this can be:

- Incorrect indication of an attachment.
- Wrong choice of attachment (undersized).
- Insufficiently supported prosthesis.
- Inaccurate, imprecise assembly.
- Divergent insertion direction of the matrix - matrix.
- Non-axial alignment of the matrix on the matrix.
- Poor oral hygiene.
- Tartar formation (encrustation in the plastic parts - emery effect).
- Use of toothpastes, especially those with a high RDA value (encrustation of particles in the plastic parts - emery effect).
- Mechanical cleaning with a toothbrush and toothpaste.
- Use of unsuitable cleaning agents (pumice, acids, etc.).
- Insufficient rinsing of the denture after cleaning.
- Inappropriate and excessive activation/deactivation.
- Activation/deactivation with unsuitable and non-original auxiliary instruments.
- Damage caused by not wearing a denture, e.g. overnight (damage caused by antagonists).
- Missing / non-executed rebasing.
- Normal, age-related wear and tear.

Clarifications/complaints can only be accepted if the batch numbers are indicated.

Visits to the dentist

Periodic, preferably annual, check-ups are necessary.

Care of dentures with attachments

Dentures with attachments should be cleaned at least once a day. The following products are recommended for cleaning:

- Denture brushes (note: dentures only, no accessories)
- Interdental brushes, dental floss, mild detergent, alkali-free soap.
- Ultrasonic device (recommended)

Toothpaste chemical compositions are adapted to enamel and contain abrasive additives and scouring agents. Therefore, toothpastes are generally not suitable for dentures (abrasion of plastic teeth, denture plastic, damage to attachments).

Do not leave dentures in the denture cleaner overnight, but at most for 15 minutes. Use only special denture brushes (only for dentures, not for attachments). After cleaning with detergents, always rinse thoroughly with water.

Patient education

Patients should be instructed on proper handling (insertion, removal of a denture), cleaning, including what cleaning agents to use/not to use, and informed of periodic checks. Patients should also be informed in writing about the accessories/materials used (designation, article number, manufacturer).

Residual risks

Please pay particular attention to the above-mentioned points. Incorrect use of attachments (wrong indication) or use in patients allergic to certain alloy components can lead to problems

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